

Bea Cool Knee Wrap

INDICATIONS FOR USE

- » Increasing medial/lateral support
- » Rheumatoid arthritis
- » Mild to moderate hyperextension
- » Mild to moderate valgus/varus deformity

MEASUREMENT

- » Circumference of Knee

FITTING INSTRUCTIONS

- » Open all the hook and loop straps
- » Wrap the brace around the leg, with the widest part at the top, making sure the hinges lie along the inside and outside of the knee, and the open section behind the knee
- » Wrap the brace around the knee and close with the hook and loop sections making sure the knee is sitting through the open section
- » Put the front straps through the D-rings and attach back on themselves to tighten the brace. The straps may need to be retightened when standing
- » Close the hook and loop straps on the back for a more custom fit

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

Size	Knee Circ	Code
Small	270 - 355	BCK/03
Medium	330 - 405	BCK/05
Large	380 - 455	BCK/07
XLarge	430 - 555	BCK/08



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR
E: info@beagleortho.com W: beagleorthopaedic.com T: 01254 268 788 f:01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.