

Elevator Sling

INDICATIONS FOR USE

- » Post trauma
- » Post-operative
- » Burns
- » Situations where arm needs to be placed in an elevated position

MEASUREMENT

- » Size is determined by appropriate length of brace

FITTING INSTRUCTIONS

- » Open all the hook & loop fastenings
- » Place the injured arm in the sling, ensuring the elbow and forearm are supported securely to avoid discomfort.
- » Close all the fastenings to secure the sling.
- » To elevate the arm use the strap, attached to the front of the sling, with the D-ring, and attach to a stand to allow elevation.
- » The brace can also be used as a high arm sling by running the shoulder strap up from the injured elbow, round the neck and over the uninjured shoulder and attach to the clip on the top of the sling.

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

Size	Codes
Small/Medium	BES/1
Medium/Large	BES/2



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: info@beagleortho.com W: beagleorthopaedic.com T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.