

Elbow Wrap

INDICATIONS FOR USE

- » Following elbow fractures and elbow surgery
- » For pain relief for elbow arthritis

FITTING INSTRUCTIONS

- » Open out the brace and place the elbow in the middle of the brace, along the stitch line.
- » Holding the brace in place, bend the elbow and wrap the straps around the arm and secure with the hoop and loop fasteners.

Tip - it is easier to attach the lower strap first.

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information

Size	Code
Universal	700/EB



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: info@beagleortho.com W: beagleorthopaedic.com T: 01254 268 788 F: 01254 268789



European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.