

Bea Comfort TLSO

INDICATIONS FOR USE

- » Spondylolisthesis

FITTING INSTRUCTIONS

- » Open the hook and loop fastenings on the front of the brace on the long shoulder straps
- » Pass the brace around the lower back with the padded section lying in the middle over the spine
- » Wrap the brace over in front and fasten with the hook and loop fastenings
- » Bring the long straps over the shoulders, moving the padding to lie on the bony part at the front of the shoulder
- » Pass the straps under the arms and cross them over at the back and back around to the front
- » Fasten them to each other using the hook and loop fastenings

WASHING INSTRUCTIONS

- » Hand wash
- » Air dry

Sizing Information


Size	Waist Circ (mm)	Code
XSmall	635 - 740	BCOM/02
Small	740 - 870	BCOM/03
Medium	870 - 990	BCOM/05
Large	990 - 1100	BCOM/07
XLarge	1100 - 1250	BCOM/08
XXLarge	1250 - 1350	BCOM/09



Beagle Orthopaedic

Prospect Building, Duttons Way, Shadsworth Business Park, Blackburn, BB1 2QR

E: info@beagleortho.com W: beagleorthopaedic.com T: 01254 268 788 F: 01254 268789

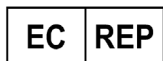

European Healthcare & Device Solutions Ltd
Sutton House, Bishopstown Road
Cork, T12 Y9TC, Ireland



Any product supplied by Beagle Orthopaedic is intended for single patient use only. Re-use of any product will invalidate the CE mark and any warranty. In case of any adverse reaction, please cease use immediately and contact your clinician for further advice.